A close-up, black and white photograph of a row of human teeth. The teeth are highly reflective, and their surfaces are covered in reflections of various natural scenes, including forests, mountains, and water. The background is dark, making the teeth stand out.

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Composite: a true alternative to ceramic!

New ideas – new indications: composite is currently experiencing a true revival. While previously only telescopic prostheses and temporary restorations were coated in polymer, the possibilities now for veneering dental materials with composite are becoming increasingly interesting – from both aesthetic and functional perspectives.

Whether for fixed restorations or removable implant restorations, modern composite is a true all-rounder. It can be processed without great effort, is biocompatible and resistant to abrasion, absorbs occlusal forces and has excellent optical properties. Dental polymers are therefore a true alternative to ceramic!

The ease with which any repairs to composite veneers can be carried out is another advantage. And the restoration adapts to any changes in the oral situation because it can be easily expanded or modified. The new Creation VC veneer composite can be easily used for a range of applications to produce naturally aesthetic results – the only prerequisite is a structured working concept that takes into account the phonetic, aesthetic and functional aspects.

Layer by layer towards dental aesthetics – either additive or subtractive.

In this brochure we take the journey together step by step – from the first wax-up to enamel layering and elaborating the dentine incisal edge contour all the way to the final restoration in red and white. There are two possible paths to an aesthetic composite restoration with Creation VC: additive or subtractive and I will present both layering variations below.

In brief, Creation VC is a modern and reliable aesthetic composite that provides us with diverse solutions in the laboratory while giving us a free hand. And those who work with a clear and systematic concept in mind can devote themselves to the multi-indication material with confidence and commitment.

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Fig. 3: The framework design was created using the wax-up but the construction can also be fully digital.

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Fig. 8: Injected fully anatomical dentine core with different flow dentines (cuspid: FD-A3.5, lateral incisors: FD-A2, central incisors: FD-A1).

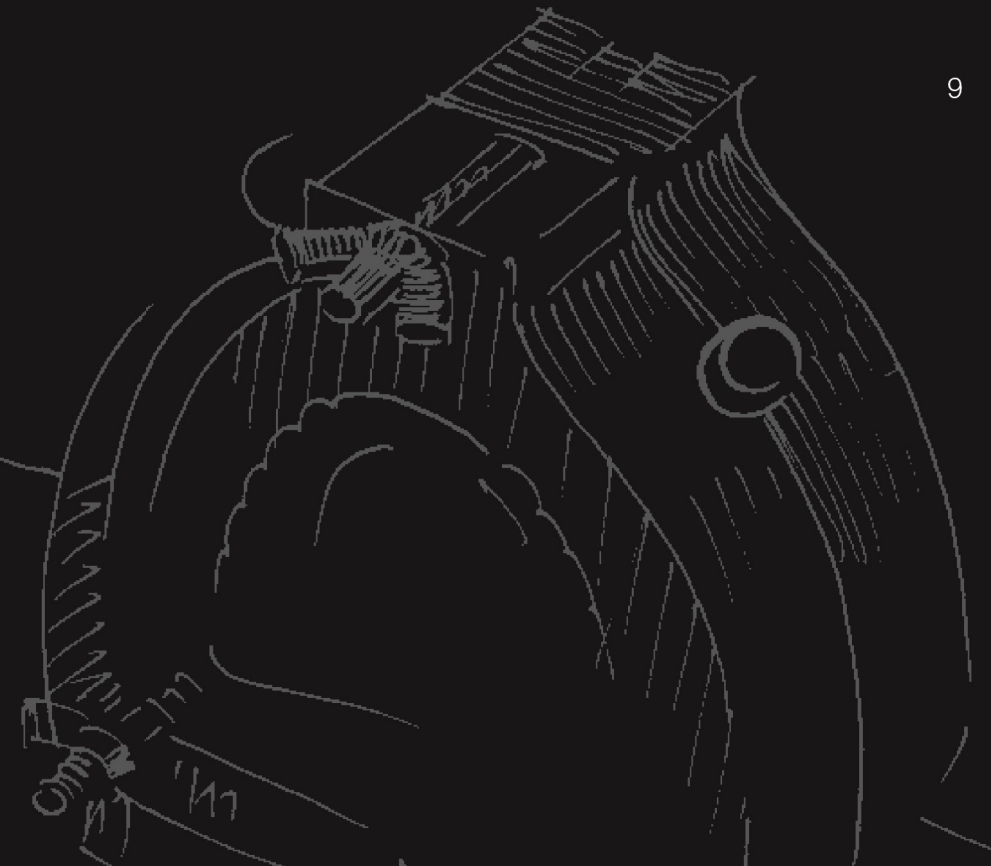
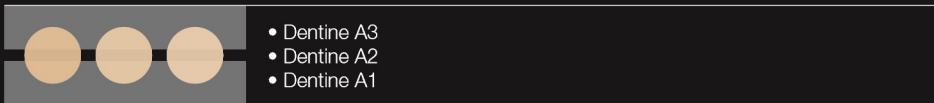
- Flow Dentine FD-A3,5
- Flow Dentine FD-A2
- Flow Dentine FD-A1

ADDITIVE SCHICHTTECHNIK / ADDITIVE LAYERING TECHNIQUE

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Fig. 9: Pressed fully anatomical dentine core with different dentine pastes (cuspids: A3, lateral incisors: A2, central incisors: A1).

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Fig. 17: Pressing over the coloured dentine core with different incisal pastes (cuspid: E-60, lateral incisors: E-59, E-58 and central incisors: E-58, E-57; each alternating).

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Fig. 18: The finished incisal layering in transmitted light reveals the wonderful opalescence of the composite material.

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Fig. 20: Camouflaging of the transition from the framework to the gingiva.

Abb. 21: Anlegen der Papillenspitzen.
Fig. 21: Creation of the tips of the papillae.

Abb. 22: Anlegen der Jugae Alveolaris.
Fig. 22: Creation of the root prominences.

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Fig. 38: Customisation with Inside Flow masses: creation of lines of Retzius, white spots and mamelon structures.

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CREATION VC – FARBKARTE / CREATION VC – COLOUR CHART

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Equipment examples

Other suitable equipment can also be used.

	GC LABOLIGHT DUO		GC STEP LIGHT SL-1		GC LABOLIGHT LV-III, II	
	Vorhärtung / Pre-curing	Aushärtung / Curing	Vorhärtung / Pre-curing	Aushärtung / Curing	Vorhärtung / Pre-curing	Aushärtung / Curing
Material / Material						
VC Opaquer (All)		1 min.				1 min.
VC Opaque Dentine VC Dentine VC Enamel VC Effect Enamel VC Transpa	10 sek. / sec.	3 min.	10 sek. / sec.		30 sek. / sec.	3 min.
VC Flow Dentine	10 sek. / sec.	3 min.	10 sek. / sec.		30 sek. / sec.	3 min.
VC Gingiva	10 sek. / sec.	3 min.	10 sek. / sec.		30 sek. / sec.	3 min.
VC Make up Neo	10 sek. / sec.	1,5 min.	20 sek. / sec.		10 sek. / sec.	5 min.
VC Composite Primer		1 min.				1 min.

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Since my dental apprenticeship in 1986 in Würzburg, I have acquired extensive experience working with M.-H. Polz in Fürth and Jan Langner in Schwäbisch Gmünd. In 1992 I completed my master training in Freiburg and since 1993 I have been working in my own laboratory in Weikersheim/Baden Württemberg.

I have been a national and international consultant for more than 25 years and I am a member of the German Association for Oral Implantology with a teaching appointment in the APW Curriculum for Implant Prosthetics since 2016.

*MDT HANS-JOACHIM LOTZ
Have fun with my suggestions!*

Vertreiber | Distributor
Creation Willi Geller International GmbH
Koblacherstraße 3, 6812 Meiningen, Austria
Tel. +43 (0)5522 76784
Fax +43 (0)5522 76784-59
info@creation-willigeller.com
www.creation-willigeller.com

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